

R38 RECEIVER

VOLUME LIMITING

The Volume Limit Feature

In certain cases where the user is not trusted to keep the receiver volume at a safe level, this feature limits the max volume a R38 will turn up to with the user controls. This could be useful for children, patients with dementia, special needs individuals, or other situations.

Turn on Volume Limiting

1. The unit needs to be set to restore the volume as last setting at every power on. Follow the steps in the R38 user guide under **Hidden Settings**.
2. Select **turn-on volume** to set the unit to restore the last setting at power on.
3. Power off the unit.
4. Power on the unit.
5. Set the volume to the desired level that will be the limit (max volume the receiver user is allowed).
6. Power off the unit.
7. Hold the **right + menu buttons** and power on the unit.
8. Press the volume increment button repeatedly to verify that when the user attempts to go above the max that the word **LIMIT** appears on the right side of the screen.
9. Power off the unit.
10. Restore an alternate turn-on volume setting again, if desired, by following the R38 user guide.

Turn Off Volume Limiting:

1. The unit needs to be set to restore the volume as last setting at every power on. Follow the steps in the R38 user guide under **Hidden Settings**.
2. Select **turn-on volume** to set the unit to restore the last setting at power on.
3. Power off the unit.
4. Power on the unit.
5. Set the volume to 0 (zero).
6. Power off the unit.
7. Hold the **right + menu buttons** and power on the unit.
8. Press the volume increment button repeatedly to verify that the volume goes above the previously set limit. The volume could go all the way up to **30** and the word **LIMIT** does not appear on the right side of the screen.
9. Power off the unit.
10. Restore an alternate turn-on volume setting again, if desired, by following the R38 user guide.