

BAT KT7

Charger for Pocketalker 2.0

QUICK SETUP GUIDE



Product Description

The BAT KT7 is designed for charging the Pocketalker 2.0 when used with NiMH batteries. It comes with adapters for various international and US outlets.

WARNING!

BEFORE CONNECTING THE CHARGER TO THE POCKETALKER, MAKE SURE THE POCKETALKER HAS NIMH RECHARGEABLE BATTERIES INSTALLED (NOT ALKALINE).

Safety Warnings


1. To prevent injury, never charge the unit in enclosed spaces such as handbags, cases, or other closed storage bags.
2. To prevent injury, unplug this device during lightning storms, or when charging is complete and the device is unused for long periods of time.
3. To prevent injury, use correct AC adapter for country of use.
4. To prevent injury, use only the power supply provided by Williams AV. Other power supplies may have similar specifications, but may not be equivalent. Use of an unapproved power supply may leave the device partially or completely inoperable, and will void the warranty.
5. To prevent injury, protect the power cord from being walked on or pinched, particularly at plugs, receptacles, and near the power jack on the unit.

WARNING!

USE THIS CHARGER WITH NiMH RECHARGEABLE BATTERIES ONLY - Williams AV P/N BAT 022 (or equivalent) AAA size NiMH, 1.2 VDC Rechargeable batteries).

Preparing the BAT KT7 charger for use

Choose the prong adapter for your country of use.

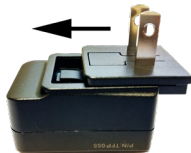

1. Slide the prong adapter onto to the top of the power supply.
2. Continue sliding down until the prong adapter clicks into a locked position.

To remove the adapter:



were putting it on.


1. Hold down the button in the middle of the prong adapter.
2. While holding down the button, slide the prong adapter off of the device the opposite direction as if you

Charging the Pocketalker

It will take about 16 hours to fully charge the batteries.

1. Open the Pocketalker battery compartment and set the battery switch to the NiMH position (Figure 1).


Figure 1: Battery Compartment

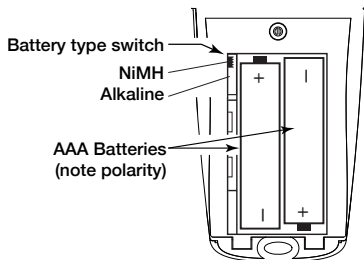

2. Insert two AAA NiMH batteries. Be sure to follow correct polarity (Figure 1). Do not use with alkaline batteries.
3. Connect the USB Cable using Figure 2:
 - a. Plug the USB "A" (larger rectangular) plug into the USB port of the power supply. Do not force it. If it doesn't slide in easily, it could be upside down. If it is, rotate it 180 degrees and try again.
 - b. Plug the USB Micro-B (smaller) connector into the charging port on the Pocketalker.

4. Follow the instructions in the previous section “Preparing the charger for use”.
5. Plug the power supply into an AC wall outlet (Figure 2).
6. The charging indicator on the Pocketalker will blink slowly while the batteries are being charged, then stay on when the batteries are fully charged.

Figure 2: BAT KT7 Charging Connection

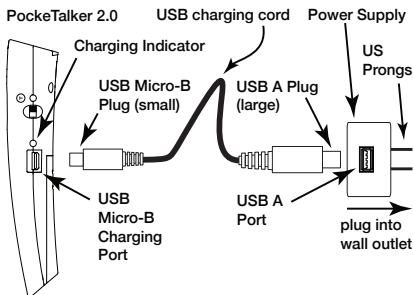

Warranty

The BAT KT7 is covered by a 90 day warranty.

Customer Service

If you experience difficulty with your system, call Toll-Free for Customer Assistance:

1-800-328-6190 (U.S.A.) or

+1 952 943 2252 (Outside the U.S.A.)

If it is necessary to return the system for service, your Customer Service Representative will give you a Return Authorization Number (RA) and shipping instructions.

WILLIAMS AV

info@williamsav.com / www.williamsav.com
800-843-3544 / INTL: +1-952-943-2252