


TELEMAX™ Phone Coupler, TEL 033

Instructions For Use & Care


INSTALLING BATTERIES:

1. Turn the elastic band inside out as shown.
2. Using your fingernail, carefully pry off the TELEMAX's *rear* panel. **DO NOT PRY OFF THE FRONT** (labeled TELEMAX).

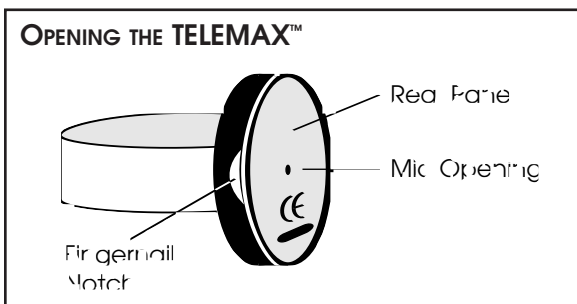

2. Locate the battery slots using the diagram on page 2.
3. Using tweezers, insert the batteries. Pay careful attention to proper polarity (+ -).
4. Carefully snap the rear panel back on to the TELEMAX.

TELEMAX™ (WITH REAR PANEL REMOVED)

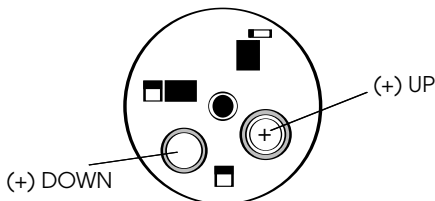


The TELEMAX's two Type 13 (BAT 028) batteries provide up to 150 hours of use, lasting up to six months on standby. Replacement batteries can be purchased from Williams Sound or your local hearing aid dispenser. Your dispenser can also install the batteries.

When replacing depleted batteries, please bring them to your dispenser or local recycling center. Improper disposal of this battery type causes harmful environmental effects.

TO USE THE TELEMAX:

1. Tap on the TELEMAX's housing with your fingertip to activate the unit.
2. Attach the TELEMAX to your phone's receiver using the elastic strip as shown.

4. Put your hearing instrument's control switch in the "T" position.
5. The TELEMAT is now ready for coupling with your T-coil. Listen to the phone signal.
6. Adjust the volume on your hearing instrument for clarity and comfort.
5. The TELEMAT will automatically turn itself off after use.

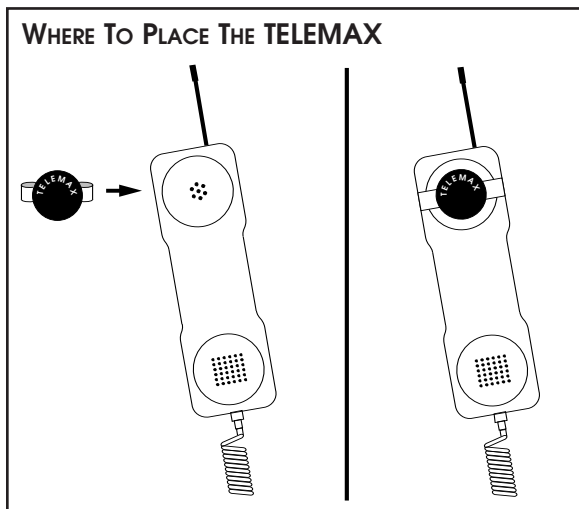



WARRANTY NOTICE

Williams Sound warrants the TELEMAX against defects in materials and workmanship for ONE (1) year. If the TELEMAX fails within the specified warranty period, Williams Sound will determine whether to repair or replace it.

This warranty does not apply to physical damage, products that have been abused or misused, or modified.

If you experience difficulty with your system, call Toll-Free for Customer Assistance:
1-800-843-3544.

If it is necessary to return the system for service, your Customer Service Representative will give you a Return Authorization Number (RA) and shipping instructions.

 **Williams Sound Corp.**
10399 West 70th St., Eden Prairie, MN 55344-3459
U.S.A. 800-328-6190 / 612-943-2252 / FAX: 612-943-2174